

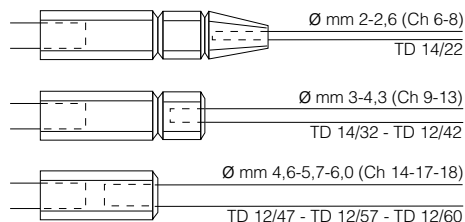
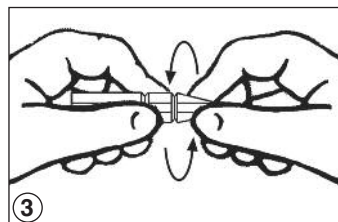
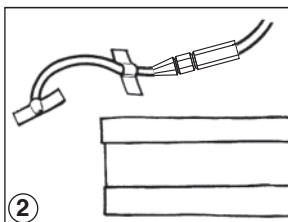
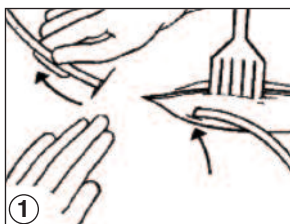
emodren®

MD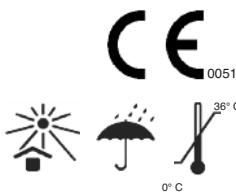

REF	
10018467	Ø 2,4 mm Ch 7
10018468	Ø 3,2 mm Ch 10
10018473	Ø 4,2 mm Ch 13
10018474	Ø 4,7 mm Ch 14
10018475	Ø 5,7 mm Ch 17
10018476	Ø 6,0 mm Ch 18
10018461	Ø 2,4 mm Ch 7 + trocar
10018464	Ø 3,2 mm Ch 10 + trocar
10018463	Ø 4,2 mm Ch 13 + trocar
10018465	Ø 4,7 mm Ch 14 + trocar
10018462	Ø 5,7 mm Ch 17 + trocar
10018466	Ø 6,0 mm Ch 18 + trocar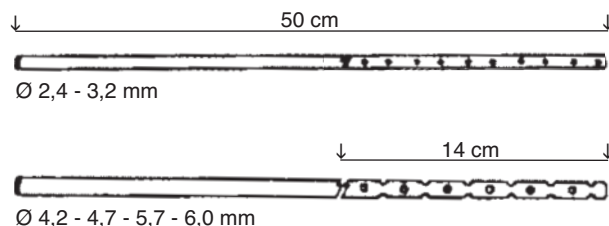

Medicalplastic s.r.l.
Via Saverio Mercadante, 15 - 20124 Milano - Italia
tel. 026704424 - Email: info@medicalplastic.it - www.medicalplastic.it
MADE IN ITALY

500900

Rev.: 0 25/11/2024

IT

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EN

PERFORATED DEHP FREE PVC DRAINAGE TUBE WITH RADIOPAQUE LINE

INTENDED USE

Tubes are a component of EMODREN, a continuous suction drainage system of the surgical wound to prevent post-operative haematomas.

DESCRIPTION

The tubes convey the drained liquids from the surgical wound to the collection bellows. The 50 cm long tubes have perforations along 14 cm (single rows of diameters measuring 2.4 mm (Ch 7) and 3.2 mm (Ch 10) – double offset rows of diameters measuring 4.2 mm (Ch 13) – 4.7 mm (Ch 14) – 5.7 mm (Ch 17) – 6.0 mm (Ch 18).

The drainage tubes can be supplied with an atraumatic disposable Inox guide-needle already assembled on the drainage tube tube: diam 2.5 mm (Ch 8) for drainage tube diam 2.4 mm diam 3.5 mm (Ch 11) for drainage tube diam 3.2 mm diam 4.5 mm (Ch 14) for drainage tube diam 4.2 mm diam 5.0 mm (Ch 15) for drainage tube diam 5.0 mm diam 6.0 mm (Ch 18) for drainage tube diam 6.0 mm

Drainage tubes are produced without intentionally added BPA.

The choice of model on the basis of patient characteristics, surgery and the correct size is up to the physician. Device intended for adult patients.

INSTRUCTIONS FOR USE

1. Introduce the drainage tube into the surgical wound from the inside to the outside by means of its guide needle of the same diameter.

The length of the perforated part must be reduced according to the size of the wound. The reference points, printed on the tubes at the distance of 50-60-70 mm from the last hole ensure that the perforated part of the tube remains inside the wound to avoid leakage from the drainage system.

Always employ strict asepsis.

Incorrect positioning of the tube could reduce its inner diameter and affect the drainage flow.

2. Fasten the drainage tube far from the wound with a single suture or adhesive tape. Connect the drainage tube projecting from the skin to the suction tube by means of the special connector, having first chosen the section of the right diameter.

3. The various sections of the connection can be removed by gripping firmly and unscrewing.

CONTRAINDICATIONS

There are no known contraindications.

WARNINGS

- Re-use of single devices may involve the following risks:
 - deterioration of the performance of the devices;
 - infection or contamination by previous use on another patient.
- The device must only be used by qualified medical staff.
- After use, the device must be treated and disposed as a hospital waste, taking all the necessary precautions and as required by current provisions of law. Sharp parts must be disposed of in special rigid containers.
- The tubes are intended for use exclusively with drainage sets manufactured by Medicalplastic. We accept no liability for any harm caused to patients by the use of 3rd party products or use other than those specifically prescribed.
- The device is supplied sterile. Do not use after the expiry date or if the packaging has been opened or damaged.
- In case of a serious incident, please promptly notify the manufacturer and your Competent Authority.
- The device has not been specifically tested on pregnant and lactating women and paediatric patients. Any use on these populations should be evaluated by the physician.